



# HOW TO ADD RACE DENTAL TO YOUR 3SHAPE TRIOS SCANNER

To add Race Dental through the **TRIOS Unite Software**, please follow below instructions.


- 1 When logged into a User, click **“More”** and select **“Store”**

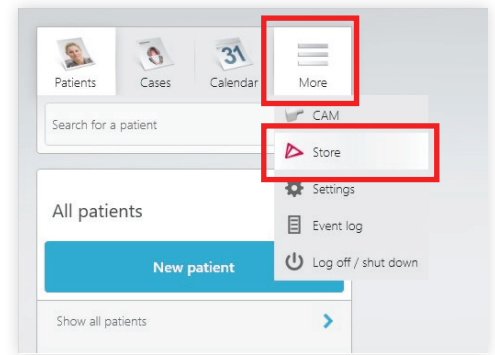

- 2 Once the store opens, search **“Race”** in the top right corner. Click **“Install”** for Race Dental.

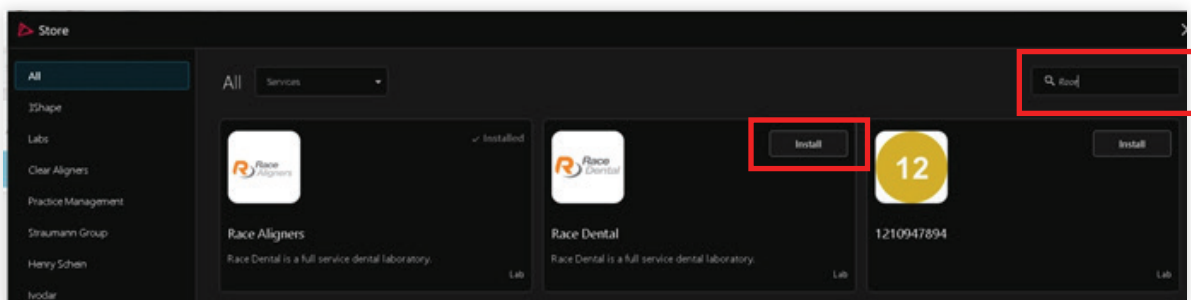


- 3 The below window will pop up for you to add a message (optional), then click **“Connect”**

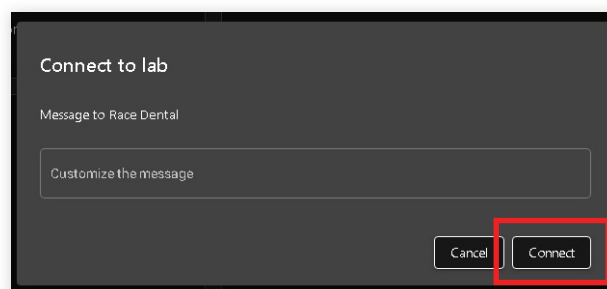


You can also to add Race Dental via the **web browser** following below instructions.

1 Please login to your 3Shape communicate account using the following link [portal.3shapecommunicate.com](https://portal.3shapecommunicate.com)

2 Select “**Connections**” from the menu at the top of the screen.

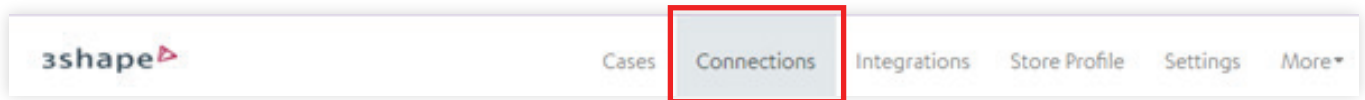

3 Select **Add Connection** Then type [trios@racedental.com.au](mailto:trios@racedental.com.au) and click **Search** (circle magnifier) then click “**Connect**”.

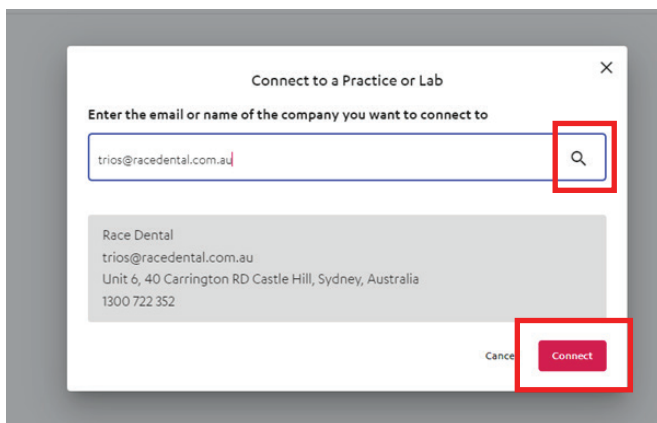

4 Once we have approved the connection Race Dental will show up on your “**Send to**” list. You may need to go to your **User profile** to **Synchronize** the connections.